**en**  
**Extension Feeding Set with ENFit™ connectors**  
**Instructions for use**

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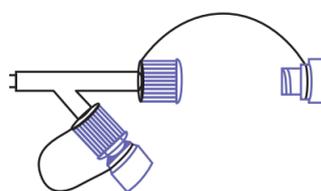
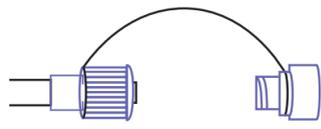
**Description:**  
 The Extension Feeding Set, with ENFit™ connectors, is comprised of a hollow tube with two different connectors at each end: an ENFit™ connector for connecting to an administration media, the other is a secure lock connector for connecting to a gastric feeding device.  
**Intended use:**  
 It is intended to provide delivery of Enteral Nutrition from administration media with ENFit™ connector and/or medication into a single patient, and for gastric decompression for a maximum of two weeks. The Extension Feeding Set with ENFit™ connectors is an accessory for a Low-Profile Gastrostomy Replacement Tube (G-Button).

**Environment and Uses**  
 The Extension Feeding Set with ENFit™ connectors is used in a hospital or home care setting.  
 - In a hospital setting, the device is operated by a nurse according to hospital guidelines and protocols.  
 - In a home care setting, home users should be either caregivers taking care of a family member or friend, or shall be Certified Nurse Assistants (CNAs) to administer nutrition and medication in a home setting. Closely related to the home users are users in a long-term care facility setting. The device is operated by a home user, after receiving training from a caregiver.


**Requirement for operation staff (home user, nurses)**  
 - Training on how to use a The Extension Feeding Set with ENFit™ connectors shall vary on institutional resources and protocols. It is expected that nurses shall receive proper training while attending nursing school or in an in-service training session. Home caregivers receive training by a health care professional.  
 - Physical ability to connect connections and operate MD.  
 - Only moderate visual impairments, Color-blindness, hearing impairment.  
 - Ability to assess the patient for any signs of pain, pressure, or discomfort.  
 - Ability to assess the patient for any signs of infection, such as redness, irritation, edema, swelling, warmth, such as rashes, or purulent or gastrointestinal drainage.  
 - Ability to look for any signs of pressure or necrosis, skin breakdown, or hypergranulation tissue in the patient.

Caution, warnings consult accompanying documents	Use by date	Keep away from direct sunlight
Manufactured by	Do not use if package is damaged	DEHP free
CE mark of compliance	Consult instruction for use	Catalogue number
Handling mark on package—KEEP DRY	Not made with natural rubber latex	UDI Unique device identifier
Single patient multiple use	Do not re-sterilize	Caution: This device is for sale by or on order of physician
QTY Quantity (not a symbol, text)	LOT Batch number	Single sterile barrier system with protective packaging outside
LENGTH Product length (not a symbol, text)	Sterilized using ethylene oxide	MD Medical Device

**Directions for Use**  
 It is highly recommended to consult instruction prior to use.  
**Warning:** For Enteral Nutrition and/or Medication Only.  
 Extension Feeding Set with ENFit™ with connectors (See Figure 4) is intended for use as an extension set to a Low-Profile Gastrostomy Replacement Tube (G-Button), incorporating safety connectors which may reduce the risk of accidental connection of an IV system to the enteral system or the enteral system to the IV system.  
**Caution:** Always be sure to wash hands thoroughly before handling feeding devices.  
 Product is sterile, do not use if package damaged or open

1. Remove the feeding port cover from the G-Button to reveal the anti-reflux valve (See Figure 2).  
 2. For gastric decompression or enteral feeding, connect the feeding set's secure lock connector (feeder) to the G-Button (See Figure 2 and 3).  
 3. Align the feeder's Connecting Tooth to the groove in the Connecting Ring. Once inside, press gently on the feeder and turn 1/4 right turn to secure the feeder in place. (See Figure 1).  
**NOTE:** If the feeder is not properly connected and secure, leakage may occur.  
 - After every use, flush the extension set with room temperature water with 10-50mL for adults or 3-5mL for children.  
 - Check for ease of flushing.  
 - After decompression or enteral feeding, flush the feeding set with room temperature water.  
 - Remove the feeding set by gently turning half a turn backwards until the Connecting Tooth can be released through the groove. (See Figure 1).

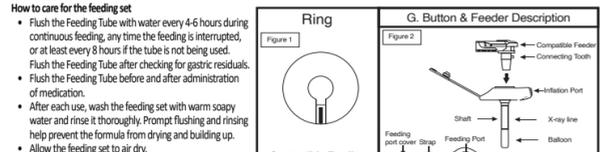**How to care for the feeding set**  
 • Flush the Feeding Tube with water every 4-6 hours during continuous feeding, any time the feeding is interrupted, or at least every 8 hours if the tube is not being used.  
 • Flush the Feeding Tube after checking for gastric residuals.  
 • Flush the Feeding Tube before and after administration of medication.  
 • After each use, wash the feeding set with warm soapy water and rinse it thoroughly. Prompt flushing and rinsing help prevent the formula from drying and building up.  
 • Allow the feeding set to air dry.

**Presentation**  
 Extension Feeding Set with ENFit™ connectors is for single patient use.  
 - For hygienic and functionality reasons (for example, if it is broken, rigid, stained), it is recommended to replace the Extension Feeding Set with ENFit™ connectors at least every two weeks.  
**Intended patient population**  
 Adult, Children older than 1 month.

**Warning:** Do not insert the connector of the Extension Feeding Set with ENFit™ into the balloon inflation port.

**Medication Administration**  
 - Administering medications should be kept separate from feeding. Any interaction of the patient's nutrients and medication should be discussed with a physician or pharmacist.  
 - Check to see whether the prescribed medication should be given on an empty or full stomach.  
 - Give liquid medication whenever possible to prevent clogging. Consult with the physician about whether it should be diluted before use.  
 - If a certain medication is available only in tablet form, consult with the physician before crushing and mixing it with water.  
 - Prior and after every use, flush the extension set with room temperature water from 10-50mL for adults or 3-5mL for children. Hydration status also influences the volume used for flushing tubes. In many cases, increasing the flushing volume can avoid the need for supplemental intravenous fluid. However, infants and individuals with renal failure and other fluid restrictions should receive the minimum flushing volume necessary to maintain unobstructed flow.

**Warning:** Never use excessive force to remove an Extension Feeding Set.  
**Warning:** Modification of the tube or any accessory parts are not allowed.  
**Warning:** Do not insert any foreign object through the tube. Consult with physician in the use of cranberry juice, carbonated soft drinks, meat tenderizer or chymotrypsin as they can actually cause clogs or create adverse reactions in some patients. If the clog cannot be removed, the Extension Feeding Set shall have to be replaced.  
**Caution:** The products should be stored in the original packaging at room temperature and in a dry environment.

**Storage**  
 The products should be stored in the original packaging at room temperature and in a dry environment.  
**Warning:** Avoid direct sunlight and heat radiation.

Applicable for the following configurations

REF	Description
EF-SSz30	EFS 30,5cm with single ENFit and straight secure lock connector
EF-SSz60	EFS 61cm with single ENFit and straight secure lock connector
EF-Rvz30	EFS 30,5cm with Y-ENFit and Right-angle secure lock connector
EF-Rvz60	EFS 61cm with Y-ENFit and Right-angle secure lock connector
EF-Svz30	EFS 30,5cm with Y-ENFit and straight secure lock connector
EF-Svz60	EFS 61cm with Y-ENFit and straight secure lock connector
EF-Rsz30	EFS 30,5cm with single ENFit and Right-angle secure lock connector
EF-Rsz60	EFS 61cm with single ENFit and Right-angle secure lock connector

1 - stands for running number; ENFit™ is a registered trademark or trademark of GEDSA. GEDSA have not sponsored or approved any SKY Medical, a.s. products.

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